

Information Sheet. Product Code:4238P Product: Padded wall mounted Shower Seat



**User weight
limit
160kgs
(25 stones)**

Should you have a problem with this device, please quote the serial number attached to the frame

DESCRIPTION:

A permanently attached shower seat, which can be lifted back against the wall, when not in use. Support legs are provided, with adjustment to suit the various heights of the users.

INTENDED PURPOSE:

As description. Will allow the user to sit in comfort and security whilst taking a shower.

INSTRUCTIONS FOR USE:

To lower, pull on the top edge of the seat downwards. The legs will automatically fold outwards into position. After use, lift the seat back into the vertical position and the device will hold in place.

DO NOT lower the seat by pulling on the legs as this will damage the stainless steel link rod and the leg itself. Warranty will be invalidated if this instruction is not adhered to.

FITTING INSTRUCTIONS:

This device must be fitted to a load-bearing wall by a competent person and fastened with expansion bolts. It is essential that the legs are adjusted for height before the seat is fastened to the wall. After fitting, the legs must be vertical. The legs may be adjusted individually by unscrewing the threaded leg and locking it into position by tightening the nut.

Use warm water and a mild household detergent when cleaning the seat.

SAFETY WARNING:

Do not stand on this device.

Ensure the user is competent to use the product.