

Information Sheet. Product Code:4240 Product:Denton Lift up Shower Seat



**User weight
limit
160kg
(25 stone)**

Should you have a problem with this device, please quote the serial number attached to the

DESCRIPTION:

A permanently attached shower seat, which can be lifted back against the wall when not in use. It has a padded seat and support legs with extensions to suit the various heights of the users.

INTENDED PURPOSE:

As description. The shower seat will allow the user to sit in comfort and security whilst taking a shower.

INSTRUCTIONS FOR USE:

No special instructions or training is necessary. To lower the seat, lift it slightly and then pivot it downwards. The legs will fall into position without aid. After use, lift the seat back to the vertical position and a simple locking device will hold it in place.

To release the seat, pull out the black retaining knob and gently lower the seat.

FITTING INSTRUCTIONS:

This device must be fitted to a load-bearing wall by a competent person and fastened with expansion bolts. It is essential that the legs be adjusted for height before the seat is fastened to the wall. After fitting, the legs must either be vertical or with the feet no more than 30mm forward from the vertical. The legs may be adjusted individually by loosening the locking nut and unscrewing the foot to the required height. Re-tighten the locking nut to maintain the required height.

SAFETY WARNING:

**Do not stand on this device!
Do not pull the seat down by the legs.**