

Information Sheet. Product Code: 4258

***Product: Adjustable
Height Shower Chair***



**Maximum user weight –
100kg (220 lbs)**

Should you have a problem with this device, please quote the batch number attached to the frame.

DESCRIPTION:

A portable shower chair which allows the user to sit in comfort whilst taking a shower. The seat is fitted with a gap front seat. The rubber ferrules on the feet will reduce slippage and marks on the shower floor.

**Height adjustable from – 48 – 60cms
Width between arms – 43cms (17")**

INSTRUCTIONS FOR USE:

No special instructions or training is required. The legs may be adjusted individually by removing the clips, sliding the extension to its new position and then replacing the clip.

SAFETY WARNING:

Make sure that the legs have all been adjusted to the same height by counting the number of holes in view in each extension leg.

This device is not suitable for standing upon or to be submerged in pools etc.