



**User Weight Limit:
190kg (30 st)**

Should you have a problem with this device, please quote the serial number attached to the frame.

DESCRIPTION:

A white plastic coated freestanding steel shower stool with perforated seat and side arms to assist in raising and lowering. The legs can be adjusted in height to suit the user.

Seat size – 15 ¼" X 10 ¼"
Height Adjustment – 18" – 24"

Base Size:

At 18" – 20" X 14"
At 24" – 21½" X 16"

INSTRUCTIONS FOR USE:

Sitting – approach the shower seat and turn around so that you are able to feel the seat with the back of legs. Put both hands on the armrests and lower yourself gently onto the shower seat. Both feet should be firmly on the ground.

Standing – Put both hands on the arm rests and push up to standing position.

The stool can be cleaned using with domestic cleaning agents and detergents. Do not use abrasive materials as this may damage the protective frame coating.

FITTING INSTRUCTIONS:

Adjust legs by gently prising out the pin clips. Slide the leg to the required height, align the holes and replace the pin clip.

SAFETY WARNING:

Ensure that the legs are adjusted to equal lengths to avoid 'rocking'.

Ensure that each of the four 'pin clips' are correctly positioned.

DO NOT stand on the stool.

The stool is designed for use in a shower environment, do not submerge for any period of time.